ClearChoice™

At Home Pregnancy Test

This pregnancy test kit contains everything you need to determine if you are pregnant. Testing is done in one simple step, and you see the result in 5 minutes. With the ClearChoice™ At Home Pregnancy Test, you can test as early as the first day of your missed period.

Sold all over the world.
Results you can trust.

Store at room temperature (59°F to 86°F).

Do not freeze.

For in vitro diagnostic use only.

Not for internal use.

Kit Contents:

This kit is equipped with everything needed for you to perform one test. Please do not open the sealed container until just before performing the test.

Before you begin the test:

1. Read the entire pamphlet carefully.
2. Your test can be performed anytime of the day. You do not need to use your first morning urine, although it is highly recommended, because if you are pregnant, your first morning urine generally contains the highest concentration of hCG. When using Phamatech's ClearChoice™ At Home Pregnancy Test, you are assured that the test will detect the presence of hCG hormone and yield the most accurate result.
3. Do not use the test beyond the expiration date printed on the package.
4. Have a watch, clock or timer available to time the test.

Urine collection:

Collect enough of your urine to fill the ClearChoice™ cup to the "FILL TO HERE" line indicated on the front of the cup. You will need about 5 minutes to complete the test. Do not shake your urine sample.


Test instructions:

Be sure that you have followed all the instructions in the "Before you begin the test" section. Failure to follow the instructions may cause an **Invalid Result**.

1. Remove the cap from ClearChoice™ cup.
2. Discard the desiccant packets.
3. Collect your urine sample in the ClearChoice™ cup enough to reach the "FILL TO HERE" line.
4. Read test results after a blue-colored spot appears in the "Test Complete" window.

Interpretation of the test:

Negative result: The presence of only one rose-pink color band within the "Read Results Here" area means that you are not pregnant.

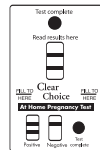Positive result: The presence of two color bands within the "Read Results Here" area means that you are probably pregnant and should consult your physician.

Invalid Result:

After performing the test (5 minutes) and no rose-pink color band is visible within the "Read Results Here" area, this result is considered invalid. The directions may not have been followed correctly or the test may have deteriorated. You have to repeat the test using a new ClearChoice™ At Home Pregnancy Test.

After Testing:

Flush the remaining urine sample down the toilet. Discard the components in a wastebasket.

Questions and Answers:

Q. When can I perform the test?

A. If you want to test on the day your period is due, it is important to use the first morning urine. The first morning urine contains the highest concentration of hCG. If your period is overdue, the use of morning urine is not as critical since your hCG level should be significantly higher.

Q. How accurate is the ClearChoice™ test?

A. Laboratory tests and consumer studies show the ClearChoice™ test to be more than 99% accurate.

Q. Can the results change after standing for a certain length of time?

A. The test is completed when a color dot appears in the "Test Complete" window, about 5 minutes, and should be read at that time. This test has a certain amount of flexibil-

(Cont'd)

ity built in with regard to its reaction time. You can read the result for up to 15 minutes after starting the test; the test should not change. After 15 minutes, a negative sample may appear to be weakly positive, therefore, any interpretation read after 15 minutes must be considered **INVALID**.

Q. Can any medicines affect the result?

A. Test results should not be affected by common pain relievers, the contraceptive pill or caffeine. However, fertility drugs may cause an inaccurate result. Only medicines that contain hCG (hCG drugs which are given by injection) can give false results.

Q. Can I get a false or misleading result?

A. This should not happen if you follow the instructions correctly. However, if you get the test device wet before using the test then you may not get a reliable result. Certain rare medical conditions can also cause a false result. If you repeatedly get unexpected results, you should check with your doctor.

Q. If the test result is positive what do I do?


A. If the test result is positive, you should see your doctor. As with all diagnostic tests, a definite clinical diagnosis should not be made based on the results of any single test. A physician must determine your status based on an evaluation of clinical and laboratory findings. Remember, early prenatal care is important to ensure the health of you and your baby.

Q. If the test is negative, what do I do?

A. A negative result means that no pregnancy hormone has been detected and you are probably not pregnant. If your period does not start within a week, you may have miscalculated the day your period was due. Repeat the test using another ClearChoice™ At Home Pregnancy Test. If the second test is negative and you still have not menstruated, you should see your doctor.

Limits of the test:

Phamatech's ClearChoice™ At Home Pregnancy Test is not reusable. The test works only if the instructions are followed precisely. Although the ClearChoice™ test is greater than 99% accurate, a few false results (positive when no pregnancy exists or negative when a pregnancy exists) can occur due to abnormal medical conditions. Check with your physician if you get unexpected or inconsistent test results. Certain health conditions such as ectopic pregnancy (pregnancy outside of the uterus) or ovarian cysts can give false results. However, oral contraceptives, painkillers, antibiotics and other common drugs do not affect the result of the test. Do not use after expiration date shown on package.

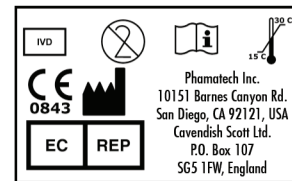Phamatech, Inc.
10151 Barnes Canyon Road,
San Diego, CA 92121
Tel: (+)1 877 466-3837 | fax: (+)1 858 635-5843
for international inquiries, please fax us at
(+)1-858-635-5843
www.phamatech.com
ClearChoice™ At Home Pregnancy Test
PHAMATECH, INC

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Phamatech Inc.